OMNI DENT®

We care. You smile.

EN Omnitemp KB

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EN Instructions for use

EN Product description:

Omnitemp KB is a fluorescent self-curing paste-paste system.

Omnitemp KB consists of base paste and catalyst paste.

Omnitemp KB is supplied in shades A2 and A3.

Insert cartridge into a compatible dispenser with a suitable piston.

Indications:

Fabrication of temporary:

- Crowns
- Bridges
- Inlays and onlays

Contraindications:

Omnitemp KB contains methacrylates, amines, terpenes, benzoyl-peroxide and BHT. Omnitemp KB should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Omnitemp KB is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Omnitemp KB should only be used by a professionally trained dental practitioner.

Use:

Usable mixing tips:

bleu / orange 4:1 / 10:1

Due to technical reasons, there may be slight differences in the fill levels of both chambers prior to initial use. You should therefore remove the cap and continue to press out the material until an equal amount has been squeezed out of both outlets. Then attach a mixing tip and lock (with a 90° turn clockwise). The material is automatically mixed by the mixing tip in the correct ratio prior to application. The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories. The attached mixing tip is intended for single use only. After use, store the cartridge tightly sealed. Check that material flows freely through the openings before reusing.

Processing directions:

First, level out any undercuts in the preparation. Cavity linings and core build-ups made from composite materials should be insulated beforehand. Phenolic substances, especially products containing eugenol and thymol, lead to curing failures from composites. The use of zinc oxide eugenol cements or other eugenol-containing materials in combination with Omnitemp KB should therefore be avoided.

Prepare an impression if prefabricated dental copings or similar restorative materials are not being used. Prepare the impression by levelling undercuts and cutting drainage channels. If the wall thickness of the temporary restoration is expected to be too thin, enlarge the impression, paying particular attention to removing the septums in the interdental embrasures. Then clean and air dry the impression. Apply Omnitemp KB directly into the impression. Do not overfill. Omnitemp KB attains an elastic phase after 1–1.5 min from beginning of mixing (approx. 0.5–1 min after insertion into the mouth).

The temporary restoration should be removed from the mouth within 1.5 min after mixing at the latest, as removal without any problems is only possible during the elastic phase. In addition, monitor the setting process by observing the excess material in the mouth. The setting time of material (in mixing tips) that has not been used cannot be compared with the setting

time in the mouth.

The working times at a room temperature of 23°C and 50% humidity are as follows. Different room temperatures may result in variations from the above times:

	mins 00:00	mins 00:30	mins 01:00	mins 01:30	mins 04:00
	Insertion in the mouth	Setting in the mouth	Removal	Complete setting	Processing

After approx. 2–3 min take the temporary restoration out of the impression and wipe dry (if necessary, use a solvent, e.g. ethyl alcohol). This removes the thin inhibition layer usually caused by oxygen and allows easier finishing. If necessary, finish the temporary restoration with fine, cross-cut carbide burs. If required, polish the temporary restoration manually (e.g. with a rubber polisher, fine sandpaper or goat hair brush).

Do not start to finish the temporary restoration until the material has completely set (approximately 4 minutes after beginning to mix). Then carefully grind any excess material and rough areas near the gingiva. Wear a mask and protective goggles and use an extraction system. Cement the temporary restoration using an eugenol-free temporary cement.

Repair of defective temporary restorations:

If the temporary restoration should break despite the extreme stability of Omnitemp KB, the following procedures are recommended:

Air bubbles: fill with Omnitemp KB.

In the case of a fracture shortly after the manufacture of the temporary restoration: re-connect the fracture with freshly mixed Omnitemp KB.

Repair of temporary restorations that have already been worn:

Roughen the fracture with a bur and place undercuts. Apply bond to the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite to the prepared areas. Reassemble the fractured pieces and cure according to the instructions for use.

Warnings, precautionary measures:

- Composite-based cavity linings and core build-ups should be sufficiently insulated (e.g. with glycerine) to avoid removal with the temporary restoration.
- Before the temporary restorations are made, the prepared cores can be insulated to prevent hypersensitivity (by using fluoride varnishes, core varnishes and the like).
- In order to avoid fractures or injuries from debris, the temporary should not be subjected to excessive loads (molar area).
- Inadequate hygiene or the intensive use of certain mouth rinses may lead to shade variations.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.
- Combination only with eugenol-free products (cements)

Storage:

Store at 4°C–23°C. Do not use after its expiry date. Protect Omnitemp KB from bright light, i.e. store in closed packages, drawers, etc.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Omnitemp KB must be reported to the manufacturer and the responsible authority.

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