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Omni Profi Varnish is a fluoride containing one component varnish. For desensitizing of teeth it is applied on dentin and enamel surface. Omni Profi Varnish contains in addition to easy soluble fluorides medium soluble Calciumfluoride and low soluble Fluorapatite. Fluorapatite particles are known to enhance remineralization of tooth material.

Omni Profi Varnish adheres to wet and dry surfaces and can be applied easily and time saving.

Omni Profi Varnish contains 22600 ppm (2,26 %) fluoride.

**Omni Profi Varnish is designed for the following indications:**

- protection and fluoridation of tooth surface
- treatment of hypersensitive teeth
- treatment of exposed dental necks
- sealing of teeth during and/or after orthodontic procedures
- sealing of tooth surface after bleaching
- Omni Profi Varnish is indicated for use in adults and in children (over 3 years old)

**Application of Omni Profi Varnish**

**Before application vigorous shake the bottle with the Omni Profi Varnish. During shaking listen to the bead inside the bottle. After shaking, the suspension should be really thin and flow easily from the dropper.**

**Recommended dosage:**

For treatment with Omni Profi Varnish the following dosage is recommended for a complete dentition:

Teeth of 1. dentition: approx. 0.2 ml (approx. 10 drops) Omni Profi Varnish

Permanent teeth: approx. 0.4 ml (approx. 20 drops) Omni Profi Varnish

**For children from 3 to 6 years ensure that not more than 0.2 ml Omni Profi Varnish is applied.**

Clean the teeth thoroughly from tartar and remove excess moisture and saliva from the areas to be treated.

Place Omni Profi Varnish in a mixing well and apply it with the supplied Micro-Applicator on the prepared teeth. The entire surface should be covered with an even thin film. Omni Profi Varnish should be applied as a thin film on the dental hard tissue. A thick film give no therapeutic advantages but peels off more easily. Leave in Omni Profi Varnish for 10-20 seconds, then dry with air.

**Close bottle immediately after use.**

**Note:**

During the first 4 hours after application of Omni Profi Varnish avoid solids, alcohol, tooth cleaning with brush and floss.

**Remarks**

- Omni Profi Varnish contains natural resins.
- In case of known hypersensitivity (allergic reactions) to natural resins, Omni Profi Varnish should not be used.
- Natural resins may inhibit the polymerization of composites and the adherence of adhesives.
- To prevent the cap to stick, wipe-off any contaminating residues on the screw of the bottle with an ethanol-containing pad.
- Omni Profi Varnish contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. After eye contact rinse out with plenty of water und call an ophtalmologist. After contact with skin, wash with plenty of water and soap.
- In patients with ulcerative gingivitis and stomatitis, Omni Profi Varnish should not be used. Omni Profi Varnish is not intended for systemic treatments.

- In case of protecting teeth with Omni Profi Varnish during application of orthodontic multi-band technique, the brackets are placed on the teeth before the application of Omni Profi Varnish.
- In case of hypersensitivity reactions, remove Omni Profi Varnish with a soft tooth brush or dental floss. Rinse with warm water.
- On the day of treatment use no other fluoride preparations. Stop treatment with fluoride tablets for several days.
- Omni Profi Varnish should not be swallowed.
- Do not use Omni Profi Varnish for children under 3 years.

**Storage:**

Do not store above 25 °C (77 °F)! Do not use after expiration date!

Omni Profi Varnish is available in bottles with 5 ml content.

**Warranty**

SanaPro Dental GmbH warrants this product will be free from defects in material and manufacture. SanaPro Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and SanaPro Dental GmbH's sole obligation shall be repair or replacement of the SanaPro Dental GmbH product.

**Limitation of Liability**

Except where prohibited by law, SanaPro Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!**

**For dental use only!**

 **Manufacturer:**  
SanaPro Dental GmbH  
D-27570 Bremerhaven  
Germany

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