Omnisept IMP

Abformdesinfektion, aldehydfrei
Disinfettante per modelli, privo di aldeide
Gietvormontsmetting, aldehydevrij

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GB Instructions for use

Type of application:
Intended for the disinfection of all dental impression materials.
Instruction for use:
Place the affected impression into the ready-to-use solution and remove again after the recommended exposure time. The impression must be completely covered with the solution. Depending on the level of contamination, the working solution may have to be replaced (RKI recommendation: replace at least once per working day). Dip alginates and hydrocolloids only briefly and allow to dry. Then rise off the impression with water. Do not mix with cleaners or other disinfectants.
Spectrum of efficacy:
Bacteria (incl. MRSA) according to EN 13727, EN 14561, VAH; yeasts (Candida albicans) according to EN 13624, EN 14562, VAH, uberculocidal (M. terrae, quantitative suspension test); Enveloped viruses (e.g. HBV, HIV, HCV) according to RKI / DVV.

Exposure times:
Bacteria (incl. MRSA) according to EN 13727, EN 14561, VAH (dirty conditions): Conc. 1 min.
Candida albicans according to EN 13624, EN 14562, VAH (dirty conditions): Conc. 1 min.
Enveloped viruses (e.g. HBV, HIV, HCV) according to RKI/DVV (dirty conditions): Conc. 1 min.
Tuberculocidal (M. terrae, clean conditions, quantitative suspension test): Conc. 15 min.

Active ingrediens per 100g:
0,24g Benzyl-C12-16-alkyldimethylammoniumchloride,
0,24g N,N-Didecyl-N,N-dimethylammoniumchloride,
0,24g C12-14-Alkyl[(ethylphenyl)methyl]dimethylammonium chloride,
0,35g Chlorhexidingluconate.

Safety guidelines:
H412 Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of the contents / container in accordance with local / regional / national / international regulations. VO 648/2004/EC: Contains < 5% cationic surfactants, fragrances

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PRISMAN GmbH